

IndeeLift®

The Human Floor Lift

For Home & Business
Self or Assisted Operation



Homes

Adult Communities


Senior Centers

Assisted Living

The People Picker Upper!
HFL-300/HFL-400

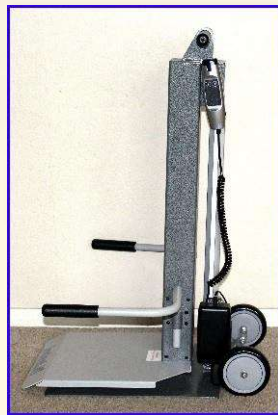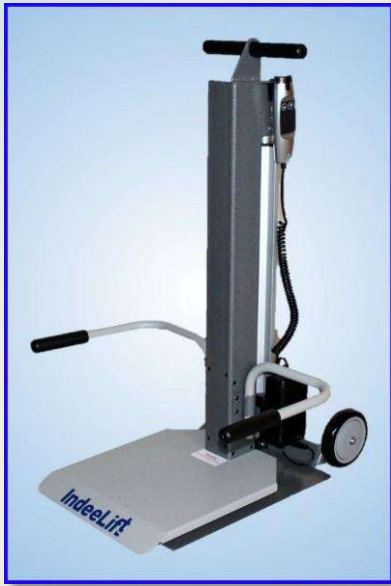

Self-Operation

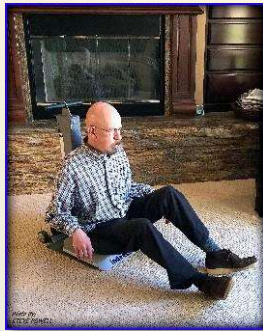

Assisted Operation





Self or Assisted Operation!

*Homes
Adult Communities
Senior Centers
Assisted Living*

The People Picker Upper! HFL-300/400

Purpose

The HFL-300 and 400 for Home & Business are designed to lift a fallen person from the floor. After a fall, a person can be easily raised without the risk of injury to themselves or a helper! In the home, the HFL allows for fall recovery without outside assistance, allowing mobility challenged individuals to remain in their own homes longer, thereby maintaining their **privacy, dignity and independence**.

Operation

Self Operation: After a fall, the fallen person scoots to the HFL, slides onto the beveled seat, pushes a button on the wired remote, activating the motor which raises them from the floor.

Assisted Operation: A helper can easily wheel the HFL to the fallen person and assist them onto the seat, if necessary.



Physical Specifications

HFL-300 HFL-400

Safe Working Load	136kg	181kg
Maximum Length	53cm	53cm
Maximum Height	132cm	132cm
External Width	38cm	38cm
Maximum Seat Height	53cm	53cm
Wheels	12.7cm	12.7cm
Power Pack	0.68kg	1kg
Total Weight	31.75kg	32.65kg

Summary of Features

- Lifts a person weighing up to 136 / 181kg. to a height of 21" above the floor
- Beveled seat-front for easy slide-on access
- Wired remote for convenient operation
- Swinging/removable Rise Assist Handles for easy transfer to wheelchair
- Self or assisted operation
- Lightweight – 23.6 / 25.4kg.
- Small footprint for tight spaces
- Powder coated for easy cleaning
- Powered by a rechargeable Lithium Ion battery pack
- Battery pack and charger included

Electrical Specifications

Standard 2-prong 110V AC Power 9' cord
Operating Environment: >+5°C <+40°C
Lithium Ion Battery: 29.9V DC 1800 mHa

T: 1300 411 973

E: sales@sasyachook.com.au

Maintain Privacy, Dignity and Independence

SASYACHOOK

5143 Tesla Road, Livermore CA 94550 USA ● (925) 456-5438 (LIFT) ● 844-700-5438 (LIFT) ● IndeeLift.com